


**BOY SCOUTS *f* AMERICA**  
**OFFICIAL FIRST AID KIT**

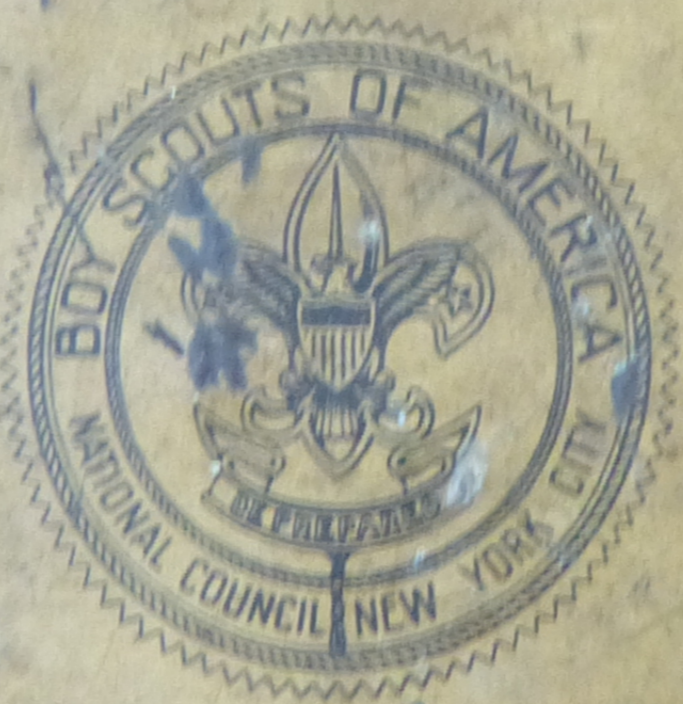

PREPARED BY

*Johnson & Johnson*  
NEW BRUNSWICK, N. J. U.S.A.

**BOY SCOUTS OF AMERICA  
OFFICIAL FIRST AID KIT CONTAINING**

- Adhesive Plaster—To hold dressings of any kind in place.
- Ammoply—For fainting, dizziness or shock. Crush in center and hold close to nostrils. Also effective when used as a swab on insect bites and stings.
- Drybak Bandaid—A protective covering for small wounds.
- Gauze Bandages (sterilized)—For binding dressings in place and as a sling in emergency.
- Halazone Tablets—For purifying contaminated drinking water. See bottle for directions.
- Joncolia (Burn Emollient)—Affords immediate relief for burns, scalds, etc. See box for directions.
- Mercuroply—Crush the tip and apply to wound. An antiseptic to prevent infection of cuts, abrasions, etc.
- Savon Cream—In case of Ivy or Oak Poison, rub cream into affected area.
- Steripads (sterilized gauze pads)—May be applied directly over a wound as an initial dressing.

Replace used articles immediately. Your druggist can supply you.



**BOY SCOUTS OF AMERICA**  
**OFFICIAL FIRST AID KIT CONTAINING**

Adhesive Plaster—To hold dressings of any kind in place.

Ammoply—For fainting, dizziness or shock. Crush in center and hold close to nostrils. Also effective when used as a swab on insect bites and stings.

Drybak Bandaid—A protective covering for small wounds.

Gauze Bandages (sterilized)—For binding dressings in place and as a sling in emergency.

Halazone Tablets—For purifying contaminated drinking water. See bottle for directions.

Joncolia (Burn Emollient)—Affords immediate relief for burns, scalds, etc. See box for directions.

Mercuroply—Crush the tip and apply to wound. An antiseptic to prevent infection of cuts, abrasions, etc.

Savon Cream—In case of Ivy or Oak Poison, rub cream into affected area.

Steripads (sterilized gauze pads)—May be applied directly over a wound as an initial dressing.

Replace used articles immediately. Your druggist can supply you.

affected area.  
Steripads (sterilized gauze pads)—May be applied directly over a wound as an initial dressing.  
Replace used articles immediately. Your druggist can supply you.

TO USE  
thumb cotton.  
AMMOPLY  
breaking tube under nose.  
Crush with inside  
JOHNSON & JOHNSON  
Especially For  
Boy Scouts of America  
MADE IN U. S. A.

AND-AID

THE  
JOHNSON  
AID  
DAGE  
son





WATERPROOF  
(VENTED)


Directions—Use one tablet to sterilize one pint of contaminated water.

Ref  
MENT  
BURNS  
SCALDS  
SUNBURN  
CHAPPING  
INSECT BITE  
INGREDIENTS  
Zinc Oxide, Phenol, Menthol,  
Oil of White Thyme, Camphor,  
Ichthyol, Lanolin, Petrolatum



ST. LOUIS  
ST-AID  
ANTISEPTIC  
DRESSING  
for  
Injuries.  
Cuts, Burns,  
Abrasions,  
Minor  
Wounds of the Skin,  
Poisonous In-  
juries.  
280z

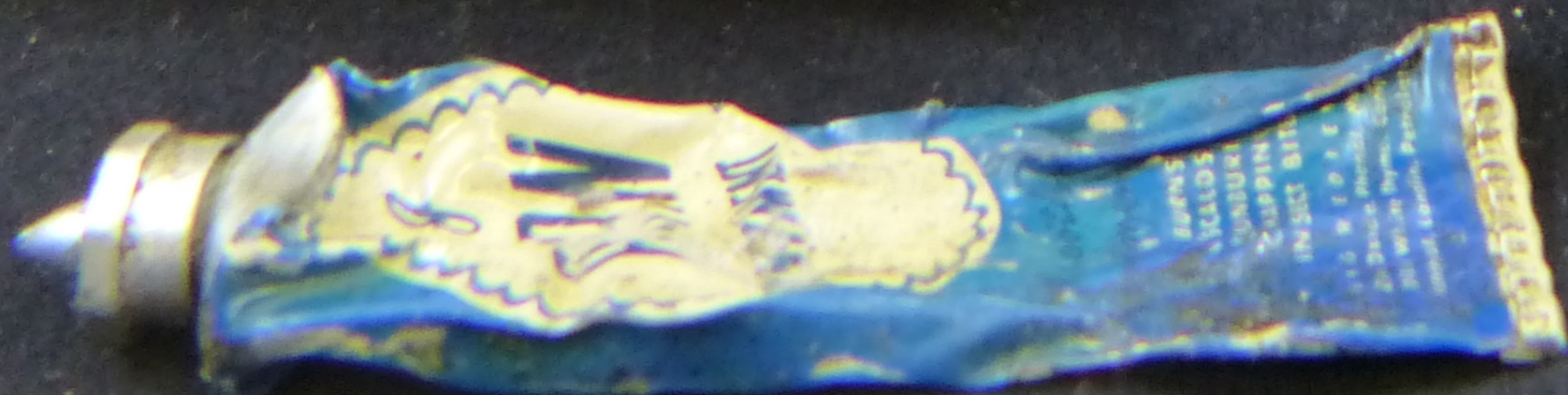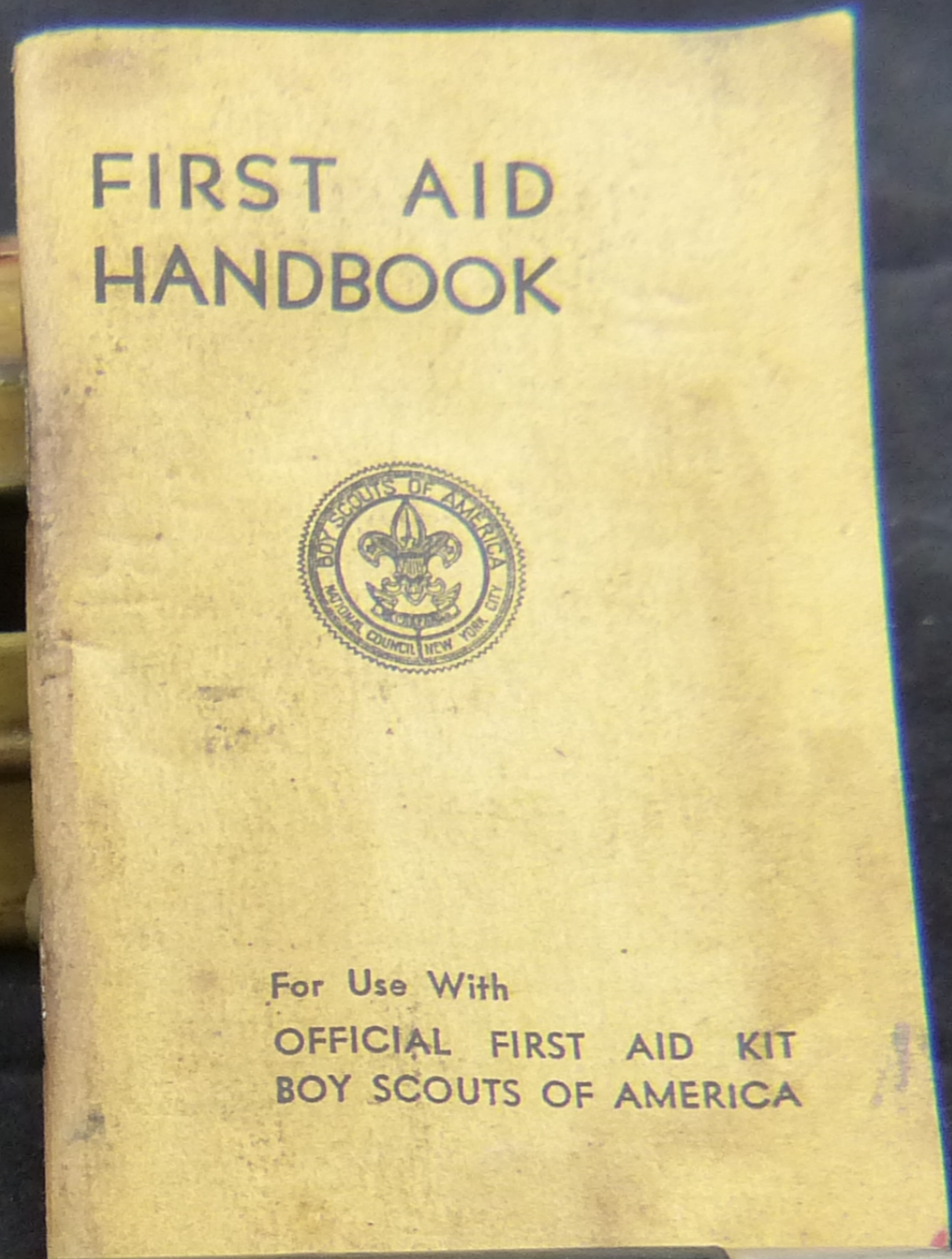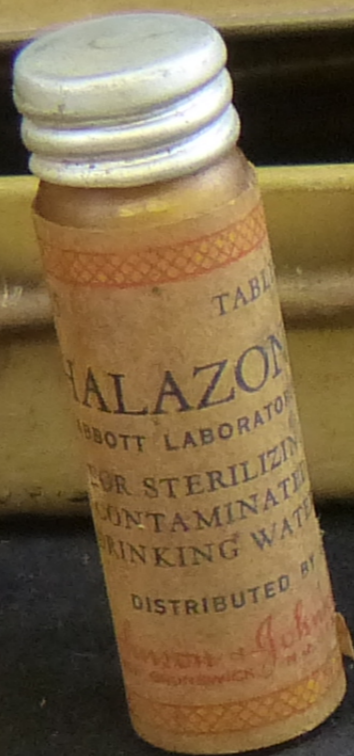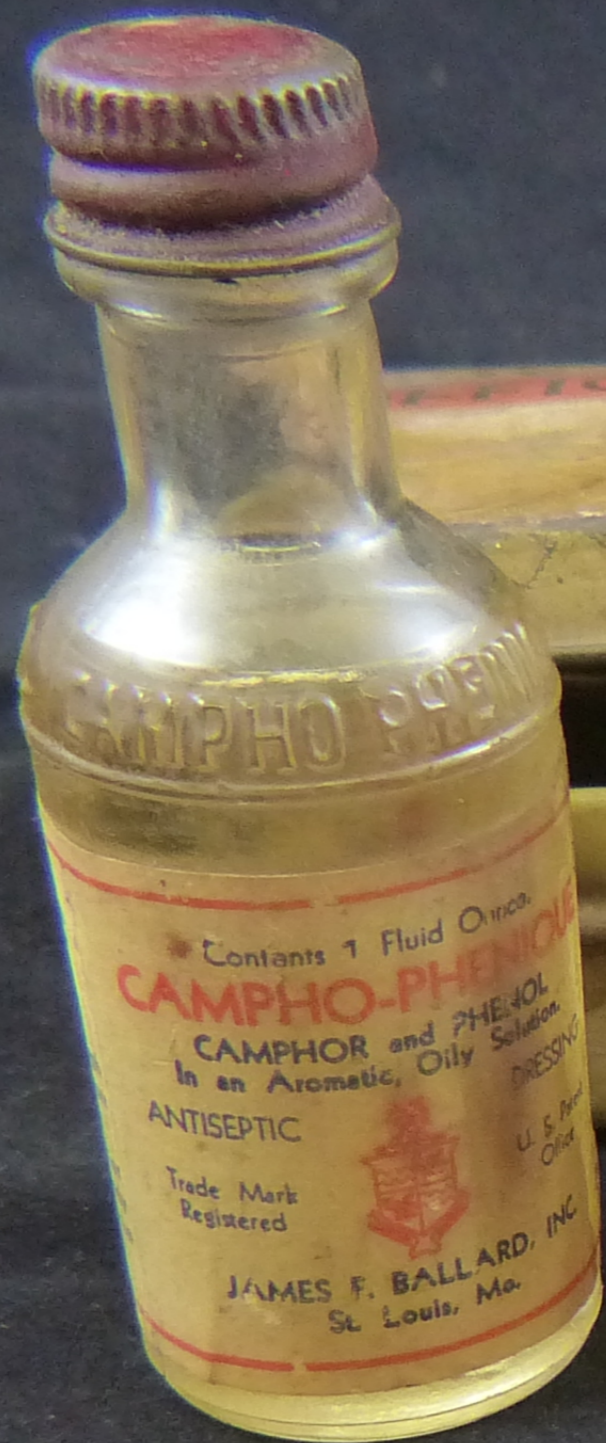

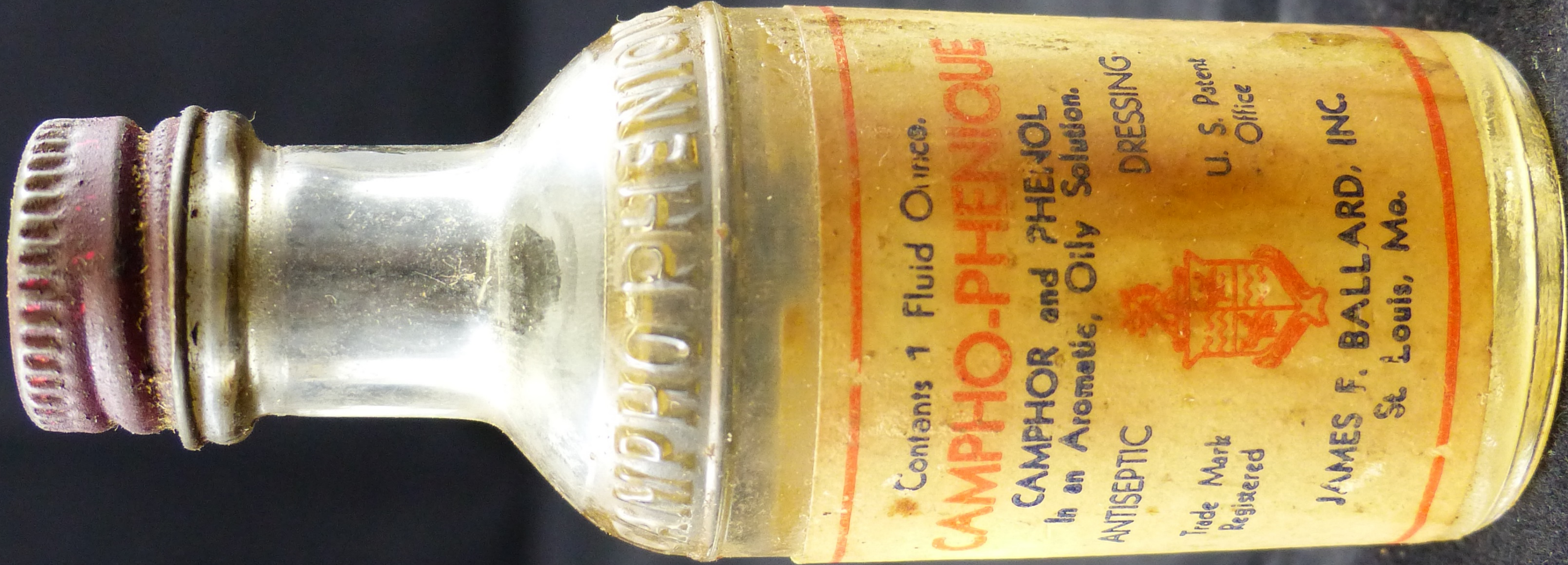

CAMPHO-PHENIQUE

Contents 1 Fluid Ounce.

**CAMPHO-PHENIQUE**

CAMPBOR and PHENOL  
In an Aromatic, Oily Solution.

ANTISEPTIC

DRESSING



Trade Mark  
Registered

U. S. Patent  
Office

JAMES F. BALLARD, INC.  
St. Louis, Mo.



MOULE & CO.

FIRST-AID  
ANTISEPTIC  
DRESSING

-for-

**Minor Injuries.**


- Minor Cuts, Burns,
- Wounds, Abrasions,
- Scratches. Minor
- Irritations of the Skin,
- Non-Poisonous In-
- sect Bites. 280z

QUE

ING

Patent  
Office

XC

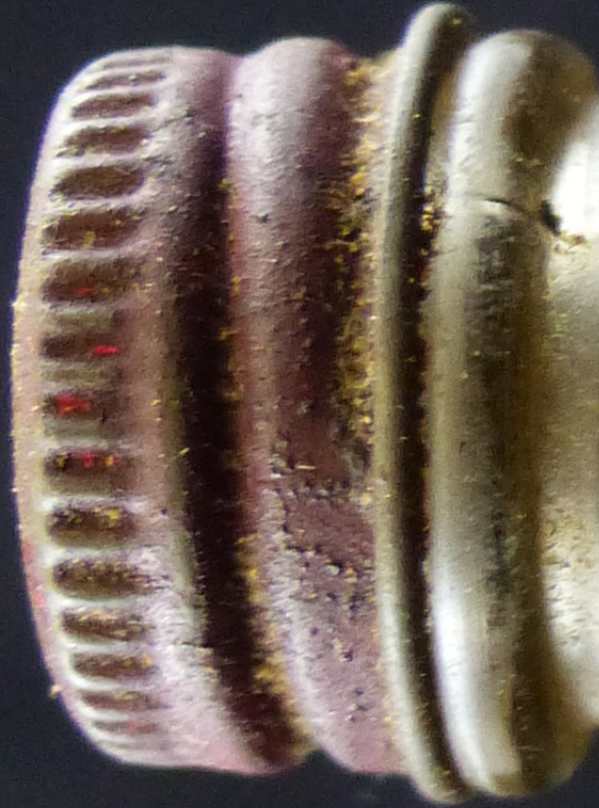

WIS-MO-CAM

For External Use.  
Never Sold in Bulk  
Accept Only the  
Original.

**DIRECTIONS**

Apply full strength  
to the affected parts  
2 or 3 times daily

For relief after  
sunburn apply only  
to small areas of the  
body at a time

CAM

ANTISE

Trade  
Registered



TABLETS

LAZONE

OTT LABORATORIES)

A STERILIZING

ANTAMINATED

DRINKING WATER

DISTRIBUTED BY

*Wm. & John Johnson*  
*ROCHESTER, N. Y. U.S.A.*

25

TAB

HALAZON

(ABBOTT LABORATORIES)

FOR STERILIZATION

CONTAINING

DRINKING WATER

DISTRIBUTED BY

*Johnson & Johnson*  
NEW BRUNSWICK, N.J.

*Directions*—Use one tab-  
let to sterilize one pint  
of contaminated water.

**BAND-AID**  
TRADE MARK

ADHESIVE BANDAGE

*Johnson & Johnson*

Plain Pad  
STERILE

WATERPROOF  
(VENTED)

**BAND-AID**  
TRADE MARK


ADHESIVE BANDAGE

*Johnson & Johnson*

(a)  
WATERPROOF

STERILE

Plain Pad

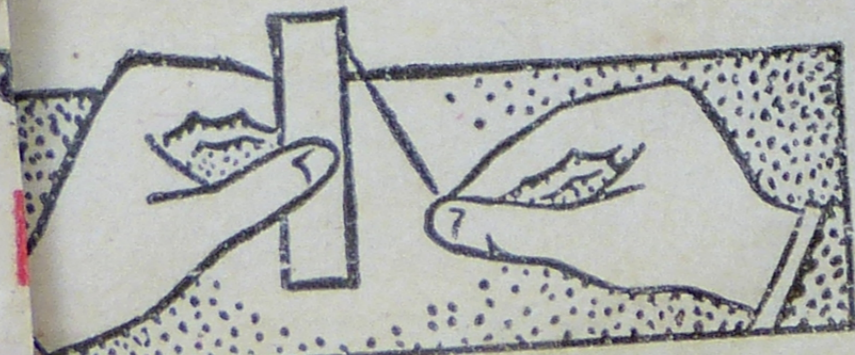

STERILITY IS NOT GUARANTEED IF ENVELOPE HAS BEEN DAMAGED OR OPENED

**TEAR OFF END →**

**PULL STRING  
DOWN**



STERILITY IS NOT GUARANTEED IF ENVELOPE HAS BEEN DAMAGED

ADHESIVE BANDAGE

**BAND-AID**

TWO

EACH

MERCUROPLY AND

AMMOPLY

*Johnson & Johnson*  
NEW BRUNSWICK, N. J., U.S.A.

TO USE AMMOPLY—Crush with  
thumb, breaking into small  
pieces.

TO USE AMMOPLY—Crush with  
thumb breaking tube inside  
cotton. Hold under nose.

Prepared by  
JOHNSON & JOHNSON  
New York, N. Y.  
The House of America

Prepared by  
**JOHNSON & JOHNSON**  
*Especially For*  
**Boy Scouts of America**

MADE IN U. S. A.

TO USE MERCUROPLY—Hold tube upright between thumb and forefinger. With other hand break tip under cotton. Turn end down. Paint over wound.

two

each





*Borax*  
**BORAX**  
**OINTMENT**

*soothe  
burns*

BURNS  
SCALDS  
SUNBURNS  
CHAPSKIN  
INSECT BITE

INGREDIENTS  
Zinc Oxide, Phenol, M...  
Oil of White Thyme, Cam...  
Ichthyol, Lanolin, Petrolin...

WATERBURY, MASS.



Johnson's Soft-Touch Baby Wipes

Protective

Apply

Liberally

Renew

Frequently



To Open:

Reverse

and puncture

with point

with point

Johnson's Soft-Touch

NEW BRUNSWICK, N.J. CHAS. CO. ILL.

MADE IN U.S.A.



